

Manufacturer: **BIONIME CORPORATION**

No. 100, Sec. 2, Daqing St., South Dist., Taichung City 40242, Taiwan

<http://www.bionime.com> E-mail: info@bionime.com

Made in Taiwan

EC REP

BIONIME GmbH

Tramstrasse 16, 9442 Berneck, Switzerland

E-mail: info@bionime.ch



101-3GP200-030
EN

BIONIME

Rightest™

Diabetes Management Software
V1.3



USER'S MANUAL

Introduction	2
PC System Requirements	3
Installation	4
Language Setting	6
Creating User Profiles	8
Connecting to PC	11
Main Features	12
Importing Data	14
Viewing Your Data	16
Data Analysis (Blood Glucose Readings)	18
Advanced Data Analysis	20
Printing Reports	28
Changing User Settings	29
Backup and Recovery	30
To-Do List	32
Un-installation	33
Customer Service	34

Introduction

The **Rightest™** Diabetes Management System is designed to help its users manage, view, and analyze blood glucose readings. It is intended for use by healthcare professionals as well as patients with diabetes.

The software included in the **Rightest™** Diabetes Management System features multiple user profiles. Each user must create a profile to login. Once logged in, users can import data from a **Rightest™** Blood Glucose Meter, view and analyze these blood glucose readings, and print blood glucose reports directly from the PC. Additionally, users can backup and recover profiles.

The **Rightest™** Diabetes Management Software is designed to help you pick out patterns in your blood glucose values. It will not provide a diagnosis or suggest treatment. Your healthcare provider may help you to find patterns and will help you make decisions about therapy based upon your blood glucose values. You should not make decisions about therapy without their help and approval. Changing your medicines without medical supervision may be dangerous.

Contents included in the **Rightest™** Diabetes Management System Kit:

1. Installation CD - Setup files for the **Rightest™** Diabetes Management Software and **Rightest™** PC Link Adapter Driver (PL2303).
2. **Rightest™** Diabetes Management Software User's Manual.
3. **Rightest™** PC Link Device.
4. **Rightest™** PC Link Instruction.

PC System Requirements

Before installing the **Rightest™** Diabetes Management Software, please make sure your system meets the following requirements:

- Microsoft Windows PC
- CPU: 550 MHz Intel Pentium 3 or above
- DRAM: 512 MB or more
- HD: 600 MB or more
- OS: Windows XP (SP3 or above), Windows Vista, Windows 7 or Windows 8
- Internet Explorer 7.0 or above
- USB 2.0 or above
- LCD screen with resolution of 1024*768 or above, and 65536 colors or more
- CD-ROM or DVD-ROM Drive
- Mouse and Keyboard
- Printer

Installation

To install the **Rightest™** Diabetes Management Software, please complete the following steps.

1. Turn on your PC.
2. Insert the **Rightest™** Diabetes Management Software Installation CD.
3. Double-click on 'My Computer'.
4. Find and double-click on your CD-Drive. By default, it should be labeled (D:) drive.
5. Double click on the "**Rightest™** DMS CD v1.3 ", and then double click on the "**Rightest™** DMS Setup v1.3.msi ". This will run **Rightest™** Diabetes Management Software Setup Wizard (shown on page 5). Follow the prompts to complete the installation. Once installation is complete, a shortcut named "**Rightest™** Diabetes Management Software" will appear on your desktop.
6. Apply the following steps ONLY when your PC Link Device which is a wired adapter, but NOT if the device uses Bluetooth technology: Go back to "My Computer" - CD-Drive (D). Double click on "PL2303_Prolific_DriverInstaller.exe" setup file. This will install the **Rightest™** PC Link Adapter Driver (PL2303). Follow the prompts to complete the installation. No shortcut will appear on your desktop for "PL2303".
7. Find the "**Rightest™** Diabetes Management Software" shortcut created on your desktop, and double-click it each time you wish to use the **Rightest™** Diabetes Management Software.



NOTE

If you are unable to upload data from your meter after the installation of PL2303 Driver is finished, please download the latest PL2303 Driver from our website www.bionime.com and reinstall it. Or please contact Bionime Customer Service for help.

Installation



NOTE

The screenshots in this User's Manual were taken from a PC using Windows 7. Appearance may vary slightly depending on the operating system of your PC.

Language Setting


After installation, when you launch the **Rightest™** Diabetes Management System Software for the first time, you will be asked to select between two different language settings: English. Select the language you need and click on " OK " to finish the setting.



NOTE
If you have chosen an unfamiliar language and wish to select another, please uninstall the software (refer to "Un-installation" instructions on Page 33) and reinstall the software.


Language Setting

If you clicked on "OK" without selecting any language, a pop-up screen will appear that says "Please Select Language". Click on "Yes" to return to the Language Setting.

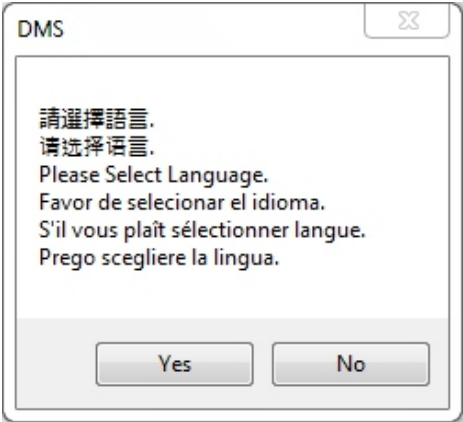

Creating User Profiles

The **Rightest™** Diabetes Management Software can be used by one or more users. To create a user profile, please complete the following steps.



1. After double-clicking on the **Rightest™** Diabetes Management Software shortcut, a welcome screen will appear.
2. Click on "Add user"  to create your user profile. Enter your personal information, meal time schedule, and blood glucose information.
3. User Profile Settings: Enter your last name, first name, date of birth, and sex.
4. Daily Meal Time Setting: You can choose from different daily meal time slots. Make sure to set the Meal Times based on your normal daily eating schedule. This allows the data analysis to accurately track the blood glucose variation between each of your meals.

MEAL TIME KEY:


- Before Bfast.: BEFORE BREAKFAST
- Before L.: BEFORE LUNCH
- Before Din.: BEFORE DINNER
- After Bfast.: AFTER BREAKFAST
- After L.: AFTER LUNCH
- After Din.: AFTER DINNER

5. Blood Glucose Limits and Ranges Setting:

The choice of values for hyperglycemia and hypoglycemia is personal and you may want to consult your health care provider to help you decide. Many people with diabetes consider hyperglycemia to be values above 180 mg/dL. For tight blood glucose control, some consider a value of 140 or even lower. Hypoglycemia is generally values less than 70, but if you do not feel symptoms of hypoglycemia, you may want to select a higher value.

6. Click on "Next"  to finish and exit the profile setup.
7. Click on "Login"  next to your user name to login.

Creating User Profiles

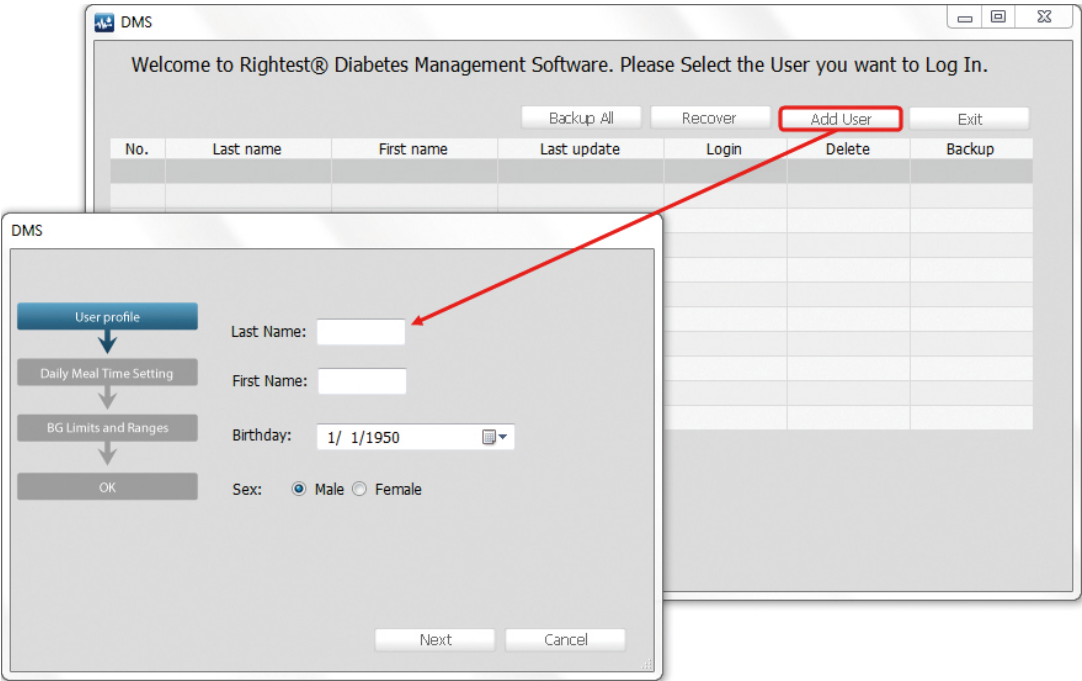

Creating User Profiles (continued)



NOTE

1. When entering your information, DO NOT leave any fields blank. You will not be able to proceed to the next step if a field is blank.
2. To ensure that all of your blood glucose readings from the meter appear in their respective meal time slots, it is extremely important that the meal schedule you entered matches your actual daily meal schedule.
3. According to the AACC (The American Association for Clinical Chemistry):
 - Hyperglycemia is any value higher than 126mg/dL.
 - The normal range for those without diabetes should have a blood glucose reading within 70~99 mg/dL.
- 4 Please discuss with a healthcare professional or physician on what values to enter for your blood glucose ranges and limits.

Connecting to PC

To connect the **Rightest™** Blood Glucose Meter to your PC, please refer to the attached "**Rightest™** PC Link Instruction".




NOTE







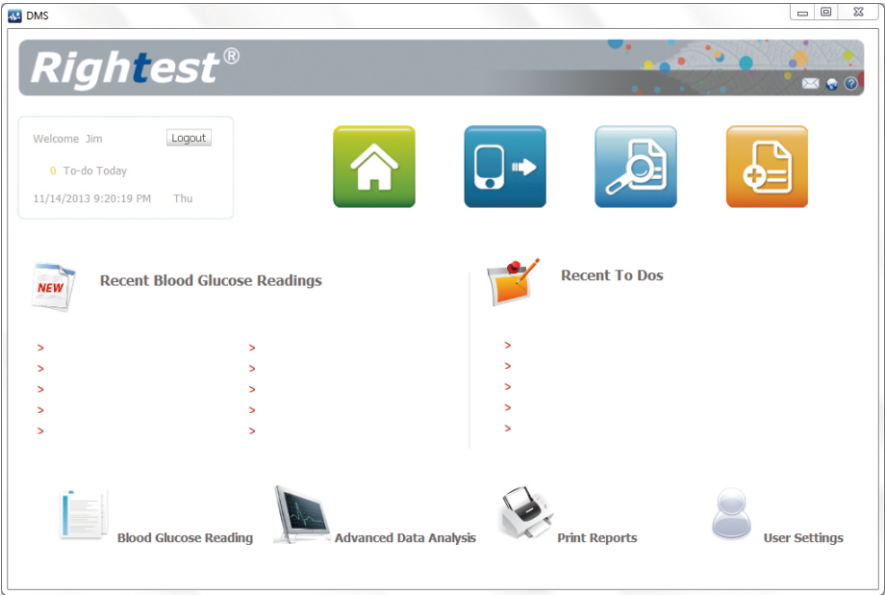

If your PC link device is a wired adapter, before connecting the meter to your PC, please make sure the **Rightest™** PC link adapter driver (PL2303) is installed. If not, please refer to the "instructions" (on Page 4) to install the driver. If the device is a Bluetooth PC Link, please make sure the connection between the Bluetooth PC Link and PC is established.

Main Features


Once you log in, the “Home Page” (pictured below) will appear on your screen. The home page will display your 10 latest blood glucose readings and your To-Do List (with the 5 most recent “To-Dos”). It will also display icons for all of the main features. The following are descriptions of each main feature.




Main Features

-  **Home Page:** Allows you to return to the Home Page.
-  **Import Data:** Uploads your blood glucose readings from your meter.
-  **Data Analysis (Blood Glucose Readings):** Provides an extensive review of your blood glucose readings.
-  **Advanced Data Analysis:** This is an extensive review of the trend chart and distribution of your blood glucose.
-  **Print Report:** Allows you to print your blood glucose analysis report and user information.
-  **User Settings:** Allows you to change your profile information.
-  **View Data:** Allows you to view your blood glucose readings.
-  **To-Do List:** Allows you to input important reminders with date, times, and alerts.


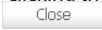
NOTE


 : This allows you to access BIONIME's services via the web or email.

 **Help:** This goes directly to the online user manual.

Importing Data

All blood glucose readings from your meter, including the date and time of the readings, will be transferred to your PC. During or after the data transfer, your readings cannot be altered or deleted. To import data, please complete the following steps.

1. Click on "Import Data"  to start transferring the data.
2. While data transfer is in progress, you should see its percentage of completion.
3. Once transfer is complete, a pop-up screen will appear that says "Import success. Total (# of data)".
Click "OK" to close the pop-up screen.
4. You can reset "Before Bfast.", "Before L.", "Before Din.", "After Bfast.", "After L." and "After Din." for each test result.(shown on page 13) by clicking the column of "Before/after meal"
5. Click on "Close"  to finish and exit.

 **NOTE**
To import your data successfully, the symbol "PcL" needs to appear on the meter's screen BEFORE you click "Import Data". If "PcL" is not showing up on the screen, please refer to the attached " **Rightest™** PC Link Instruction".

Importing Data

DMS

Com Port: COM1
Model: GM720
State: Import success.

[Start over](#)
[Print Reports](#)
[Close](#)

Status	Analysis	Date/Time	Blood glucose (mg/dL)	Before/after meal	Notes
P!	<input checked="" type="checkbox"/>	7/1/2013 5:39:00 PM	161	Before Din.	<P!> Sport.
II	<input checked="" type="checkbox"/>	7/1/2013 7:14:00 PM	93	Before Bfast. Before L.	<I!> Illness.
SI	<input checked="" type="checkbox"/>	7/1/2013 10:52:00 PM	253	Before Din.	<SI> Special.
P!	<input checked="" type="checkbox"/>	7/2/2013 7:47:00 AM	334	After Bfast. After L.	<P!> Sport.
II	<input checked="" type="checkbox"/>	7/2/2013 9:16:00 AM	80	After Din.	<I!> Illness.
SI	<input checked="" type="checkbox"/>	7/2/2013 12:53:00 PM	88	Before L.	<SI> Special.
P!	<input checked="" type="checkbox"/>	7/2/2013 2:11:00 PM	81	After L.	<P!> Sport.
II	<input checked="" type="checkbox"/>	7/2/2013 5:38:00 PM	198	After Din.	<I!> Illness.


Footnote:
<CI> A Control Solution Test Result.
<TI> A BG result that was tested outside of the normal temperature range.
<DI> The BG result that is not included in the average calculation.
<HI!> Higher than the highest measurable value.
<LO!> Lower than the lowest measurable value.
<@!> Time error.
<P!> Sport.
<I!> Illness.
<SI> Special.

Viewing Your Data

The *Rightest™* Diabetes Management Software allows you to review your blood glucose readings for the previous 90 days. The data table and trend chart will be displayed on the Home Page.


The data table will provide measurement dates, time stamps, and meal times. You can choose from six different preset time periods to view your data.

On the trend chart, there will be two lines that represent the hyperglycemia and hypoglycemia limits. The normal blood glucose range lies between these two values. The limits and range can help you better understand your blood glucose readings. To view your data, please complete the following steps.


1. Click on “View Data” . The software will automatically display blood glucose readings from the 30-day time period.
2. You may select a different preset time period, located to the left of the data table.

Viewing Your Data









1. Click on  to enlarge the trend chart. To exit zoom view, simply close the window.
 2. Make sure to consult with a healthcare professional or your physician before entering the hyperglycemia and hypoglycemia limits, as well as the normal blood glucose range.
 3. On the Trend Chart:
 - The gray area in between the limits indicates the normal blood glucose range.
 - The purple line is the hyperglycemia value.
 - The blue line is the hypoglycemia value.
- If you move your mouse cursor over the Trending Line, you can see your Blood Glucose Readings.

Data Analysis (Blood Glucose Readings)

Click on the “Blood Glucose Readings”  icon to view and analyze your blood glucose readings.

Instructions for Data Analysis:

- You can choose from seven different time periods: 1 day, 7 days, 14 days , 21 days, 30 days, 60 days, or 90 days. You may also enter a customized time period.
- You can choose from nine different meal times: Before/After Breakfast, Before/After Lunch, Before/After Dinner, Before/After Meal, or View All Data.
- Your blood glucose readings will be represented by different colors:
 -  [Red] This blood glucose reading is higher than the hyperglycemia value
 -  [Orange] This blood glucose reading is considered to be a high outlier
 -  [Green] This blood glucose reading is within the normal range
 -  [Blue] This blood glucose reading is considered to be a low outlier
 -  [Indigo] This blood glucose reading is lower than the hypoglycemia value
 -  [White] An ERROR has occurred in the data
- The software has five different methods to view your data with statistical analysis: View All Data, Total Analysis Data, Average Value, Standard Deviation, and Coefficient of Variation. In the table, you can view the percentage of each blood glucose reading.
- When analyzing your data, you can select particular readings to view. Keep in mind that unselected readings will be excluded from ALL calculations and reports in Data Analysis and Advanced Data Analysis.

Data Analysis

DMS

Customize

From: 7/ 1/2013 To: 11/30/2013

Before/after meal

☒ All ☐ Before Bfast. ☐ After Bfast. ☐ Before Meal ☐ Before L. ☐ After L. ☐ After Meal ☐ Before Din. ☐ After Din.

<input checked="" type="checkbox"/> Hyperglycemia Value	463	48.4 %	All Data	1000
<input checked="" type="checkbox"/> High Outlier	226	23.6 %	Data to be analysed	956
<input checked="" type="checkbox"/> Normal Value	185	19.4 %	Data excluded	0
<input checked="" type="checkbox"/> Low Outlier	51	5.3 %	Average	196.4mg/dL
<input checked="" type="checkbox"/> Hypoglycemia Value	31	3.2 %	Standard deviation	89.04
<input type="checkbox"/> Erroneous data	44		Coefficient of variation	0.45

Any Data with Errors will NOT be analyzed. You can Select specific Data that you want to analyze.


No.	Date/Time	Blood glucose	Before/After meal	Notes	Analysis
1	7/1/2013 12:21:00 PM	216 mg/dL	Before L.		<input checked="" type="checkbox"/>
2	7/1/2013 2:40:00 PM	148 mg/dL	After L.		<input checked="" type="checkbox"/>
3	7/1/2013 5:39:00 PM	161 mg/dL	Before Din.	<PI> Sport.	<input checked="" type="checkbox"/>
4	7/1/2013 7:14:00 PM	93 mg/dL	After Din.	<II> Illness.	<input checked="" type="checkbox"/>
5	7/1/2013 10:52:00 PM	253 mg/dL	After Din.	<SI> Special.	<input checked="" type="checkbox"/>
6	7/2/2013 7:47:00 AM	334 mg/dL	Before Bfast.	<PI> Sport.	<input checked="" type="checkbox"/>
7	7/2/2013 9:16:00 AM	80 mg/dL	Before Bfast.	<II> Illness.	<input checked="" type="checkbox"/>
8	7/2/2013 12:53:00 PM	88 mg/dL	Before L.	<SI> Special.	<input checked="" type="checkbox"/>
9	7/2/2013 2:11:00 PM	81 mg/dL	After L.	<PI> Sport.	<input checked="" type="checkbox"/>
10	7/2/2013 5:38:00 PM	198 mg/dL	After Din.	<II> Illness.	<input checked="" type="checkbox"/>
11	7/2/2013 7:31:00 PM	212 mg/dL	After Din.	<SI> Special.	<input checked="" type="checkbox"/>
12	7/2/2013 10:41:00 PM	109 mg/dL	Before Din.	<PI> Sport.<II> Illness.	<input checked="" type="checkbox"/>
13	7/3/2013 7:29:00 AM	202 mg/dL	Before Bfast.	<PI> Sport.<SI> Special.	<input checked="" type="checkbox"/>
14	7/3/2013 9:44:00 AM	183 mg/dL	Before Bfast.	<II> Illness.<SI> Special.	<input checked="" type="checkbox"/>
15	7/3/2013 12:57:00 PM	151 mg/dL	After L.	<PI> Sport.<II> Illness.	<input checked="" type="checkbox"/>



NOTE

Data in the meter with an ERROR will be transferred to the database, but will be excluded from all calculations and analysis.


Advanced Data Analysis

Click on the "Advanced Data Analysis" icon  to view the Advanced Data Analysis Report. Select among "Blood Glucose Trend", "Blood Glucose Distribution" and "Standard Day".


Instructions for Blood Glucose Trend Chart:

1. You can view data based on different time periods: 1 day, 7 days, 14 days, 21 days, 30 days, 60 days, and 90 days. You can also input a customized time period.
2. You can view data based on different meal times: Before/After Breakfast, Before/After Lunch, Before/After Dinner, Before/After Meal, or View All Data.
3. The trend chart can display either a regression curve or a move average line. The regression curve lets you see the long-term patterns of the selected time period. The move average line allows you to view short-term changes within the selected time period.

NOTE




1. You can uncheck "Point Value" from your readings if you do not wish them to be displayed. If so, only the minimum and maximum point values will be shown. When you move your mouse cursor along the trend line, it will show the date, time, and point value of that particular reading.
2. If you only select one day to view your readings, then the regression curve and moving average line will NOT be displayed.
3. Click on  to enlarge the trend chart. To exit zoom view, simply close the window.
4. On the trend chart:
 - The gray area in between the limits indicates the normal blood glucose range.
 - The purple line is the hyperglycemia value.
 - The blue line is hypoglycemia value.

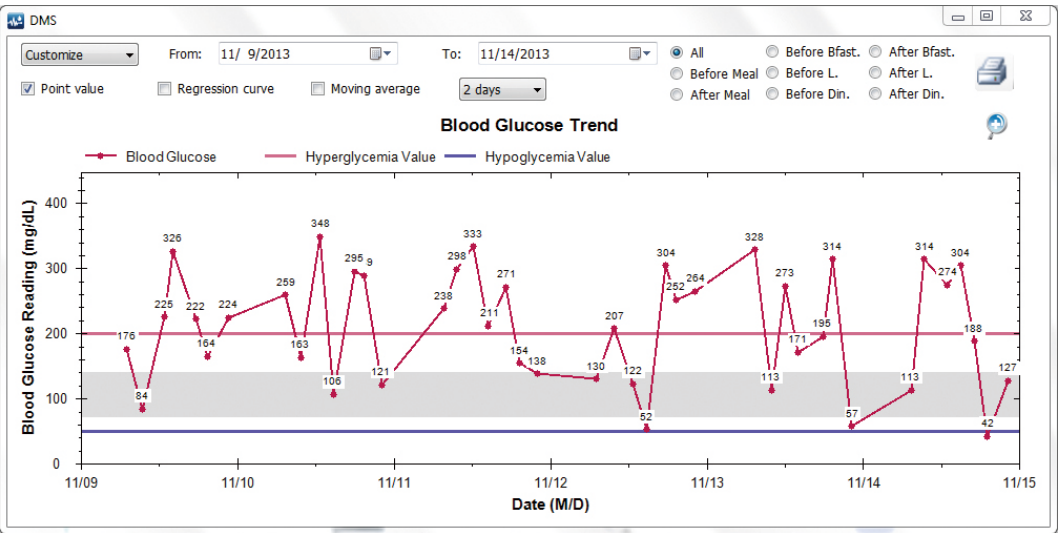
Advanced Data Analysis



Advanced Data Analysis (continued)

The trend chart can be printed in one page. Please complete the following steps.

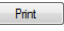1. Click on  icon for the Print Preview.
2. Click on  icon in the preview page.
3. Click on "Print"  to print the trend chart.

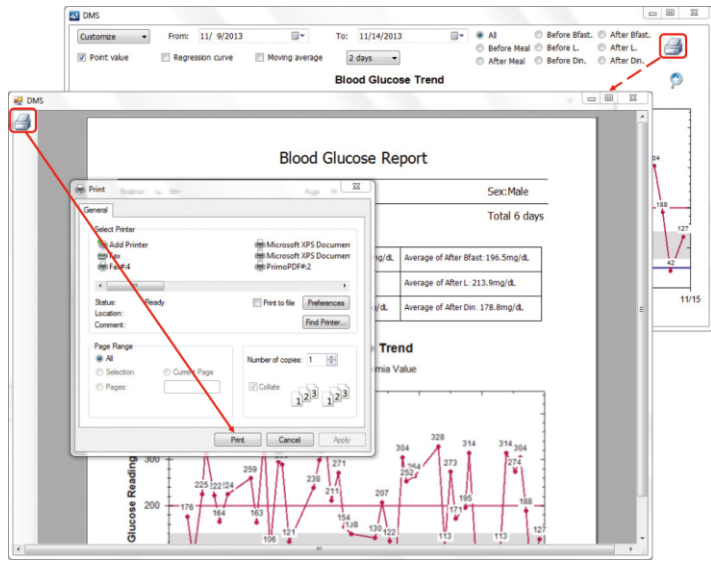


Advanced Data Analysis (continued)

The **Rightest™** Diabetes Management Software allows you to view your blood glucose readings in different formats, including pie chart. However, the percentage of each blood glucose reading is not available in the pie chart.

Instructions for Pie Chart:

1. You can display certain blood glucose readings by selecting the different time periods: 1 day, 7 days, 14 days, 21 days, 30 days, 60 days, or 90 days. You may also input a customized time period.
2. In the pie chart, your blood glucose readings will be represented by different colors. The following is a COLOR KEY for the values.
 - [Red] The blood glucose reading is higher than the hyperglycemia value.
 - [Orange] The blood glucose reading is considered to be a high outlier.
 - [Green] The blood glucose reading is within the normal range.
 - [Blue] The blood glucose reading is considered to be a low outlier.
 - [Indigo] The blood glucose reading is lower than the hypoglycemia value.

Advanced Data Analysis (continued)

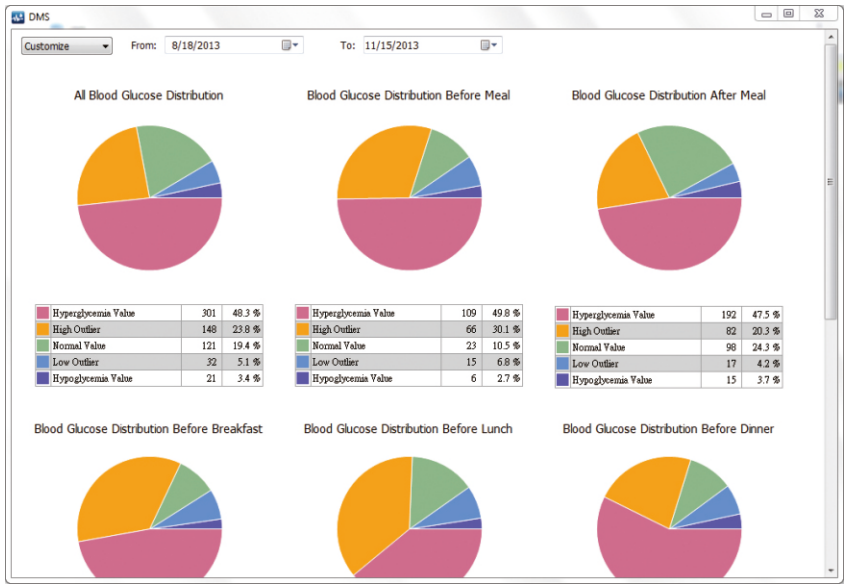


NOTE
Any errors within the data will NOT be included in the pie chart.

Advanced Data Analysis (continued)


Introductions for Standard day chart:

1. You can view data based on different time periods: 1 day, 7 days, 14 days, 30 days, 60 days and 90 days. You can also input a customized time period.
2. You can view data based on different meal times: Before/After Breakfast, Before/After Lunch, Before/After Dinner, Before/After Meal, or View All Data.
3. The standard day chart can display the 24-hr profile of the readings within the selected time period.

NOTE




1. You can uncheck "Point Value" from your readings if you do not wish them to be displayed. When you move your mouse cursor to a reading, it will show the date, time, and point value of that particular reading.
2. Click on  to enlarge the trend chart. To exit zoom view, simply close the window.
3. On the standard day chart:
 - The gray area in between the limits indicates the normal blood glucose range.
 - The purple line is the hyperglycemia value.

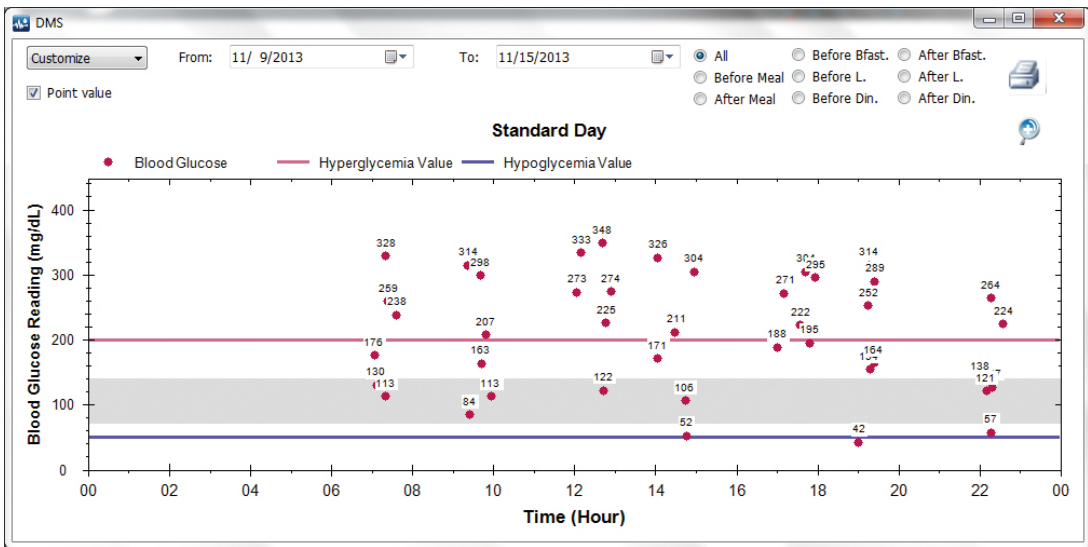
Advanced Data Analysis (continued)



Advanced Data Analysis (continued)

The standard day chart can be printed in one page. Please complete the following steps.

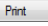
1. Click on  icon for the Print Preview.
2. Click on  icon in the preview page.
3. Click on "Print"  to print the standard day chart.

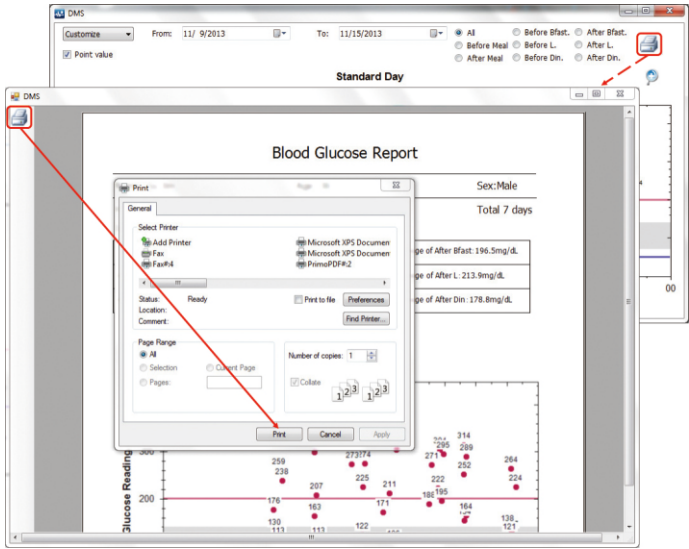


Printing Reports

The **Rightest™** Diabetes Management Software allows you to print your blood glucose readings and reports.

Printing Steps:



- 1. Click on “Print Report”  to enter the printing settings.
- 2. Select the desired time period for the readings and reports you wish to print.
- 3. You can click on ‘Advanced Print’ for more options.


 **NOTE**

- 1. If you wish to print the regression curve and the moving average line, use the “Advanced Print” function.
- 2. If you select data from one specific day, the regression curve and the moving average line in the trend chart will NOT be available for printing.

Changing User Settings

After you log in, you can update or change your user profile information using the following step:

- 1. Click on “User Settings”  to access your user profile.
- 2. You can modify your personal information (except name), meal time schedules, and blood glucose limits and ranges.
- 3. Click on “Save and Close”  to save your settings.

 **NOTE**


- 1. It is important that the meal time schedule you set up in the **Rightest™** Diabetes Management Software is the same as your daily meal schedules.
- 2. If any fields are left blank or entered incorrectly, the software will not allow you to continue. If you are unsure, click “Recommended Setting”.
- 3. According to the AACC (The American Association for Clinical Chemistry), the suggested hyperglycemia maximum is any reading higher than 126mg/dL, and the normal blood glucose range is any reading within 70~99 mg/dL.
- 4. Be sure to consult with a healthcare professional or your physician BEFORE entering in your blood glucose limits

Backup and Recovery

You can back up one or more user profiles at the welcome screen before you login. To backup and recover, please complete the following steps.

Backup Steps:


1. Click on "Backup" to backup your individual user profile, or click on "Backup All" to backup all user profiles.
2. Select the folder you wish to save the backup in.
3. Click on "OK" to finish backing up.

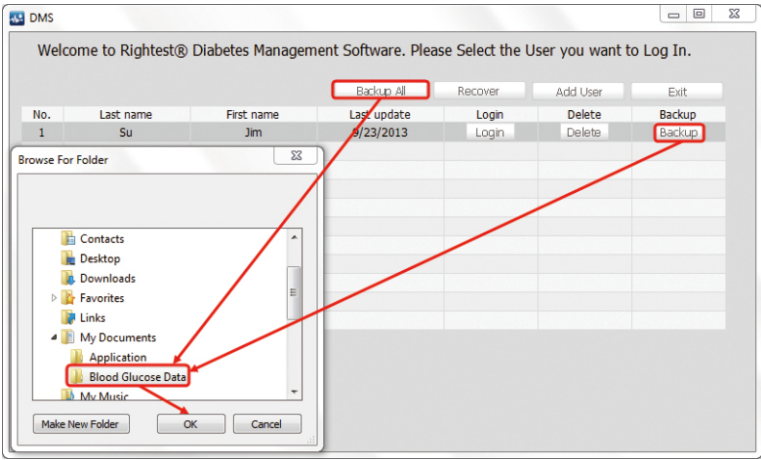

Backup and Recovery

Recovery Steps:


1. Click on "Recover" to retrieve your previously saved data.
2. Select the file you wish to recover.
3. Click on "Open" to finish the recovery.

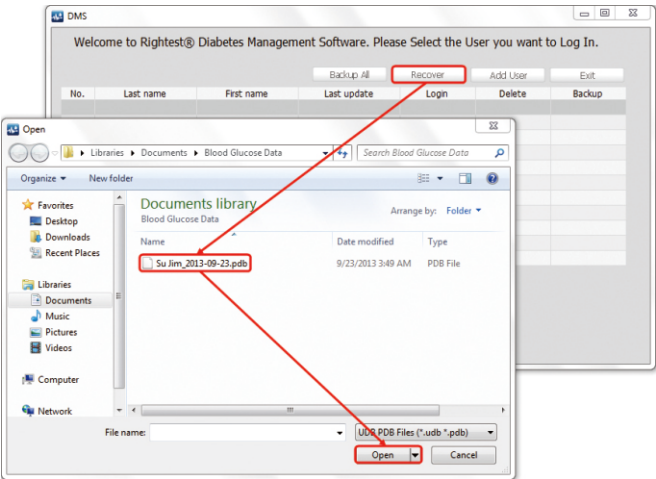




NOTE
Once you recover data, your current profile will be replaced with the backup profile you selected.

To-Do List

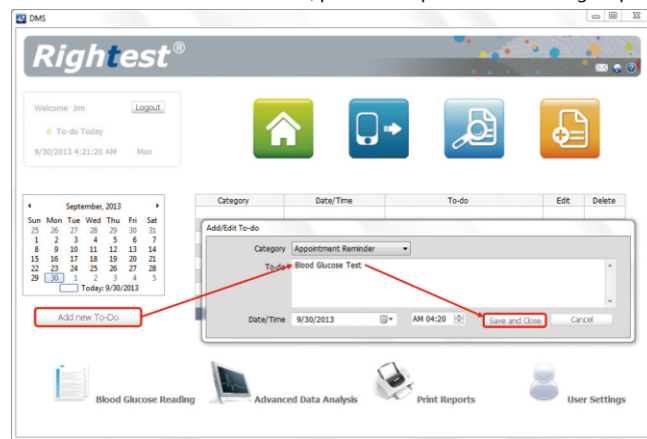With the **Rightest™** Diabetes Management Software, you can set up reminders of upcoming events, medicine dosage or important comments and alerts. To create a To-Do list, please complete the following steps.



1. Click on "To-Do List" .
2. Click on "Add new To-Do"  to create a new event.
3. Input the time, select the category of the reminder, and enter in any notes about the event.
4. Click on "Save and Close"  to finish and save.


NOTE

1. In each event, the maximum number of characters in a note is 255. If you exceed this amount, an ERROR message will pop up.
2. All reminder messages will alert you at their appointed time.

Un-installation

To un-install the **Rightest™** Diabetes Management Software, please complete the following steps.

1. Go to the "Control Panel" and select "Uninstall a program".
2. Find and select " **Rightest™** Diabetes Management Software".
3. Click on "Uninstall", then confirm by clicking "Yes" to un-install the program.

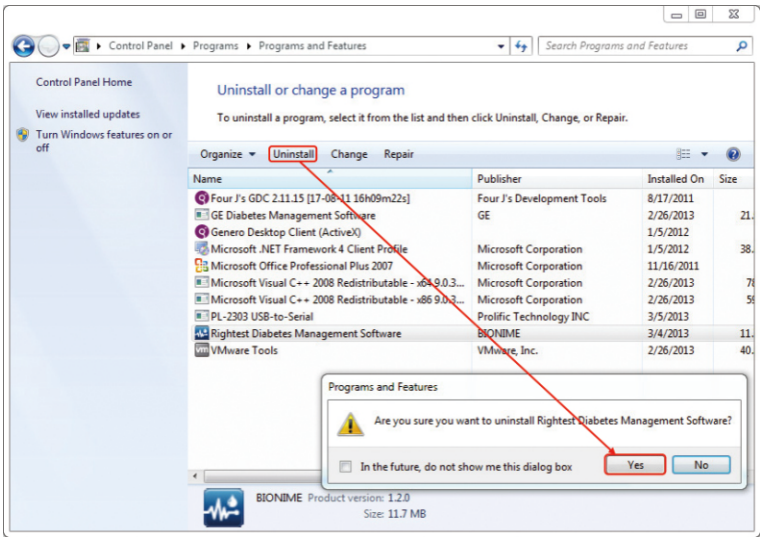

Customer Service

Please be sure to read and review ALL instructions of this User's Manual prior to using the *Rightest*™ Diabetes Management System.

We sincerely like to provide complete, considerate services to our customers. Please review all the instructions to make sure you are performing the steps correctly. If you have any questions or in case of problems with the products, please contact your local Bionime customer service.